


How to install exocad libraries

Right click on **DentalCADApp.exe** -> Open file location

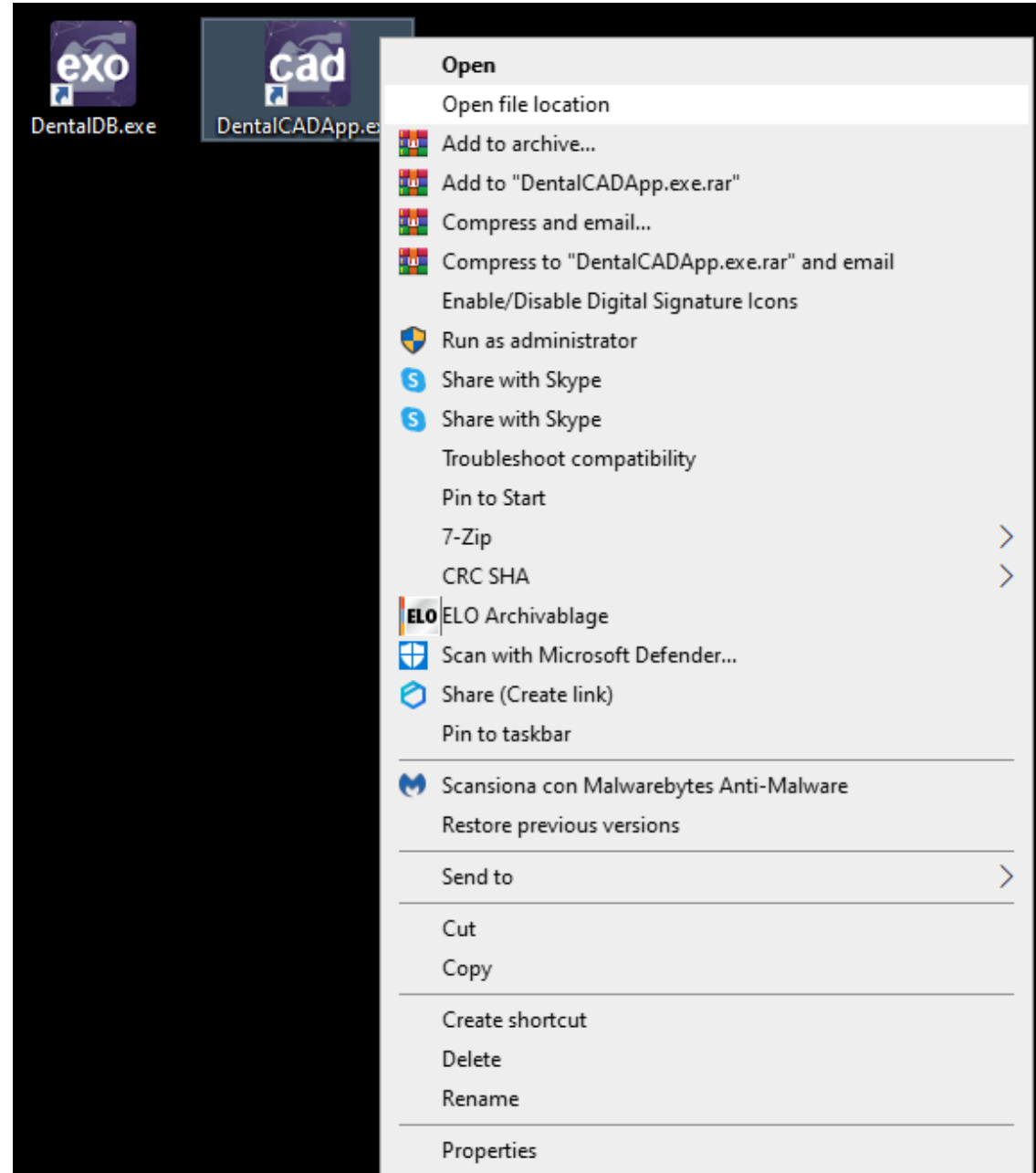

Go to the folder **DentalCADApp**

The screenshot shows a Windows File Explorer window with the address bar set to `This PC > Desktop > Library > exocad-DentalCAD3.0-2020-12-23 > DentalCADApp > bin`. The `DentalCADApp` folder name in the address bar is highlighted with a red box. The main pane displays a list of files and folders with columns for Name, Date modified, Type, and Size. The left sidebar shows navigation options like Quick access, Tresorit Drive, Synced Tresors, OneDrive, This PC, and Network. The bottom left corner indicates 60 items.

Name	Date modified	Type	Size
OpenSource.dll-LICENSES	27/12/2020 20:34	File folder	
prerequisites	23/01/2020 09:55	File folder	
AForge.Video.DirectShow.dll	27/12/2020 21:05	Application exten...	67 KB
AForge.Video.dll	27/12/2020 21:05	Application exten...	28 KB
AlphaFS.dll	27/12/2020 21:05	Application exten...	289 KB
DentalBaseDicom64.dll	27/12/2020 20:34	Application exten...	2,853 KB
DentalBaseDotNet.dll	27/12/2020 21:05	Application exten...	23,458 KB
DentalBaseExternals64.dll	27/12/2020 20:34	Application exten...	128,376 KB
DentalCADApp.exe	27/12/2020 21:05	Application	7,467 KB
DentalCADApp.exe.config	27/12/2020 21:11	XML Configuratio...	2 KB
DentalCADDongleActivator.exe	27/12/2020 20:35	Application	1,839 KB
DentalCADEexternals64.dll	27/12/2020 20:34	Application exten...	161 KB
DentalCADScannerControls.dll	27/12/2020 21:05	Application exten...	291 KB
DentalConfig.dll	27/12/2020 21:05	Application exten...	1,563 KB
DentalConfig.XmlSerializers.dll	27/12/2020 21:06	Application exten...	264 KB
DentalConfigControl.dll	27/12/2020 21:05	Application exten...	75 KB
DentalData.dll	27/12/2020 21:05	Application exten...	1,902 KB
DentalData.XmlRoots.dll	27/12/2020 21:06	Application exten...	4 KB
DentalData.XmlRoots.XmlSerializers.dll	27/12/2020 21:06	Application exten...	1,825 KB
DentalData.XmlSerializers.dll	27/12/2020 21:06	Application exten...	2,476 KB
DentalHelperLib.dll	27/12/2020 21:05	Application exten...	1,734 KB
DentalHelperLib.XmlSerializers.dll	27/12/2020 21:06	Application exten...	209 KB

Open the folder **library**

The screenshot shows a Windows File Explorer window titled 'DentalCADApp'. The address bar indicates the path: This PC > Desktop > Library > exocad-DentalCAD3.0-2020-12-23 > DentalCADApp. The left sidebar shows 'This PC' selected. The main pane displays a list of files and folders. The 'library' folder is highlighted and circled in red.

Name	Date modified	Type
bin	16/02/2021 17:41	File folder
config	27/12/2020 20:56	File folder
exocam	27/12/2020 21:08	File folder
icons	27/12/2020 20:56	File folder
languages	27/12/2020 20:56	File folder
library	31/03/2021 13:45	File folder
plugins	27/12/2020 21:08	File folder
Skins	27/12/2020 21:06	File folder
DentalCADApp-generic-2020-12-27-integrity.sha256	27/12/2020 21:11	SHA256 Fil
DentalCADApp-generic-2020-12-27-integrity.sha256.sig	27/12/2020 21:11	SIG File

10 items | 1 item selected

Open the folder implant

The screenshot shows a Windows File Explorer window with the following details:

- Address Bar:** This PC > Desktop > Library > exocad-DentalCAD3.0-2020-12-23 > DentalCADApp > library
- Navigation Pane (Left):** Quick access, Tresorit Drive, Synced Tresors, OneDrive, This PC (selected), Network
- Main Content Area:** A table of folders with columns for Name, Date modified, Type, and Size. The 'implant' folder is highlighted and circled in red.
- Status Bar (Bottom):** 24 items | 1 item selected


Name	Date modified	Type	Size
articulator	27/12/2020 21:08	File folder	
artiregister	27/12/2020 21:09	File folder	
attachments	01/10/2020 15:29	File folder	
bar	15/12/2020 09:24	File folder	
bolts	23/01/2020 09:50	File folder	
bridgesplitter	23/01/2020 09:50	File folder	
controls	27/12/2020 20:35	File folder	
gfy	27/12/2020 21:09	File folder	
implant	09/07/2021 09:47	File folder	
metadata	27/12/2020 21:09	File folder	
modelcreator	27/12/2020 21:11	File folder	
movementregister	23/01/2020 09:54	File folder	
nesting	23/01/2020 09:54	File folder	
ortho	27/12/2020 21:06	File folder	
pontics	27/12/2020 21:09	File folder	
production	27/12/2020 21:09	File folder	
prosthetictoothpresets	21/12/2020 16:24	File folder	
prosthetictoothsets	03/05/2021 10:44	File folder	
rendereffects	27/12/2020 20:35	File folder	
retentions	23/01/2020 09:55	File folder	
smiledesign	27/12/2020 20:35	File folder	
surgicalguide	03/05/2021 10:44	File folder	

Leave the folder **implant** opened


File Explorer window showing the **implant** folder. The address bar shows the path: This PC > Desktop > Library > exocad-DentalCAD3.0-2020-12-23 > DentalCADApp > library > implant. The main pane displays a list of 40 folders, all named **MegaGen_...** with various specifications like **SHA_Cust_Abt_H_Height_7mm**. The left sidebar shows **This PC** selected. The bottom left corner indicates **40 items**.

Name	Date modified	Type
MegaGen_AR_SHA_Cust_Abt_H_Height_7mm	10/03/2021 15:47	File folder
MegaGen_AY_Cust_Abt_9mm	10/03/2021 15:47	File folder
MegaGen_AY_Cust_Abt_13mm	10/03/2021 17:08	File folder
MegaGen_AY_Ext_R_Cust_Abt_9mm	10/03/2021 15:47	File folder
MegaGen_AY_Ext_R_Cust_Abt_13mm	10/03/2021 15:47	File folder
MegaGen_AY_Ext_S_Cust_Abt_9mm	10/03/2021 15:47	File folder
MegaGen_AY_Ext_S_Cust_Abt_13mm	10/03/2021 15:47	File folder
MegaGen_AY_Ext_W_Cust_Abt_9mm	10/03/2021 15:47	File folder
MegaGen_AY_Ext_W_Cust_Abt_13mm	10/03/2021 15:47	File folder
MegaGen_AY_Onestage_Cust_Abt_10mm	10/03/2021 15:47	File folder
MegaGen_AY_SHA_Cust_Abt_H_Height_4mm	10/03/2021 15:47	File folder
MegaGen_AY_SHA_Cust_Abt_H_Height_5mm	10/03/2021 15:47	File folder
MegaGen_AY_SHA_Cust_Abt_H_Height_7mm	10/03/2021 15:47	File folder
MegaGen_MN_Cust_Abt_9mm	10/03/2021 15:47	File folder
MegaGen_MN_Cust_Abt_13mm	10/03/2021 15:47	File folder
MegaGen_MU_N_Type_Cust_Abt_13mm	10/03/2021 15:47	File folder
MegaGen_Octa_Cust_Abut_11mm	10/03/2021 15:47	File folder
MegaGen_STR_R_Cust_Abt_9mm	10/03/2021 15:47	File folder
MegaGen_STR_R_Cust_Abt_13mm	10/03/2021 15:47	File folder
MegaGen_STR_S_Cust_Abt_9mm	10/03/2021 15:47	File folder
MegaGen_STR_S_Cust_Abt_13mm	10/03/2021 15:47	File folder


Select the 3 zip files and right click on them -> Extract Here

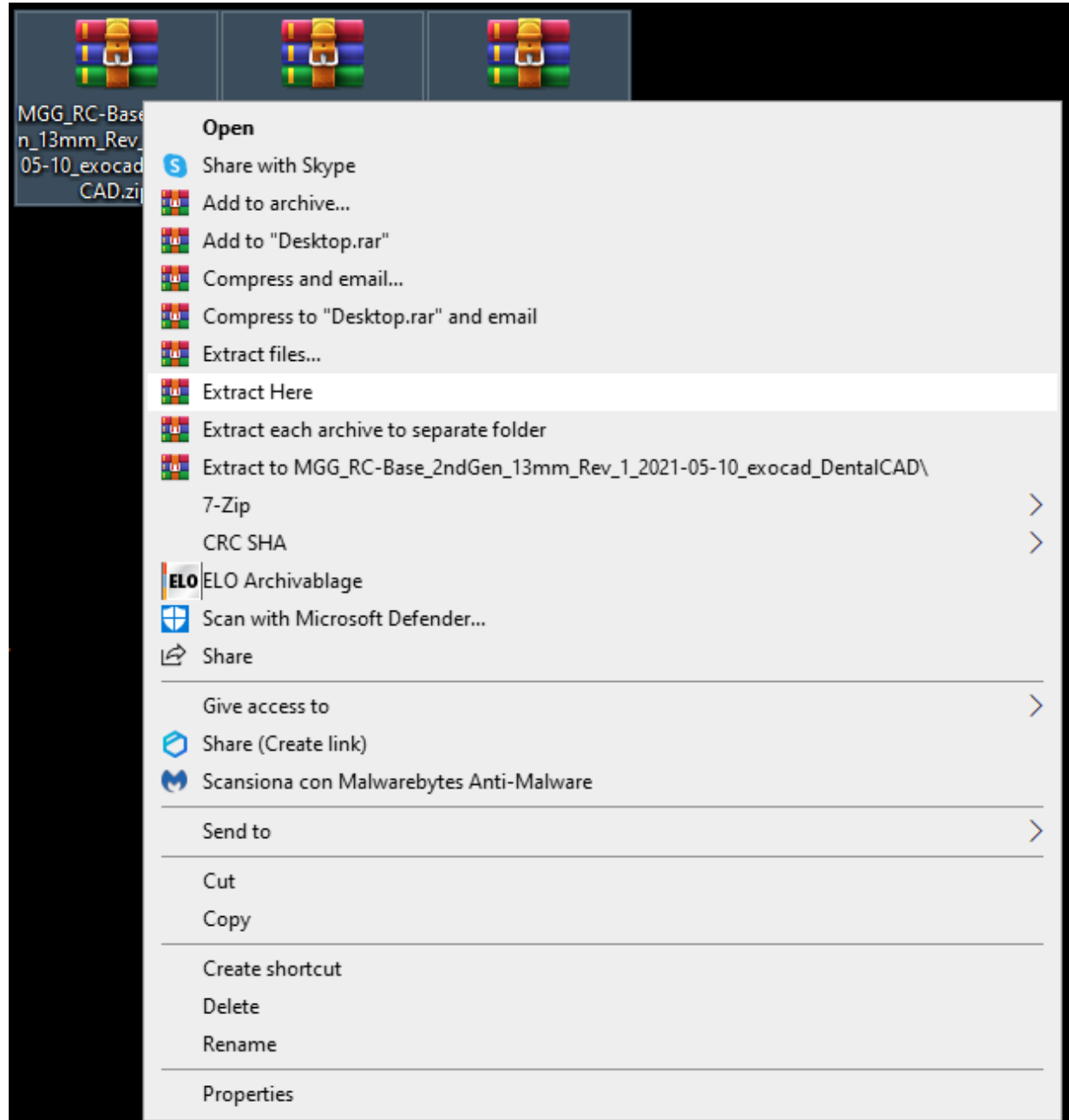

Move the 2 folders containing the DentalCAD libraries...

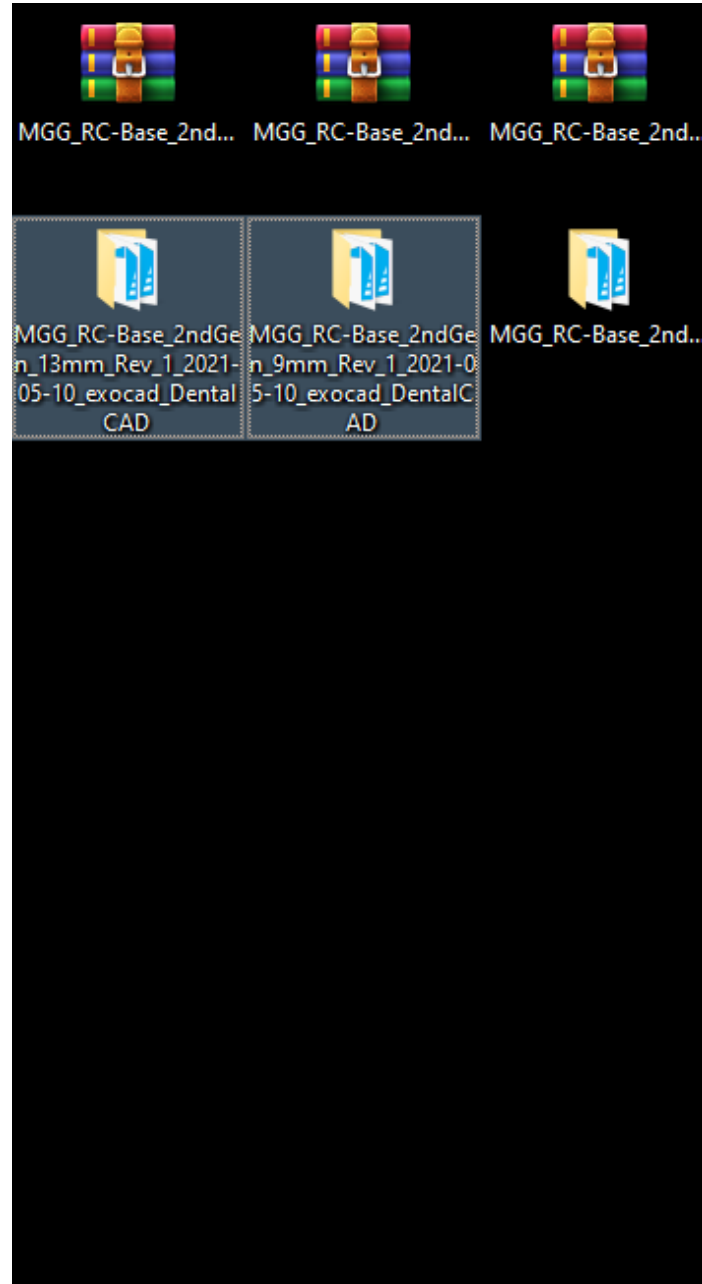

... in the **implant** folder left opened before

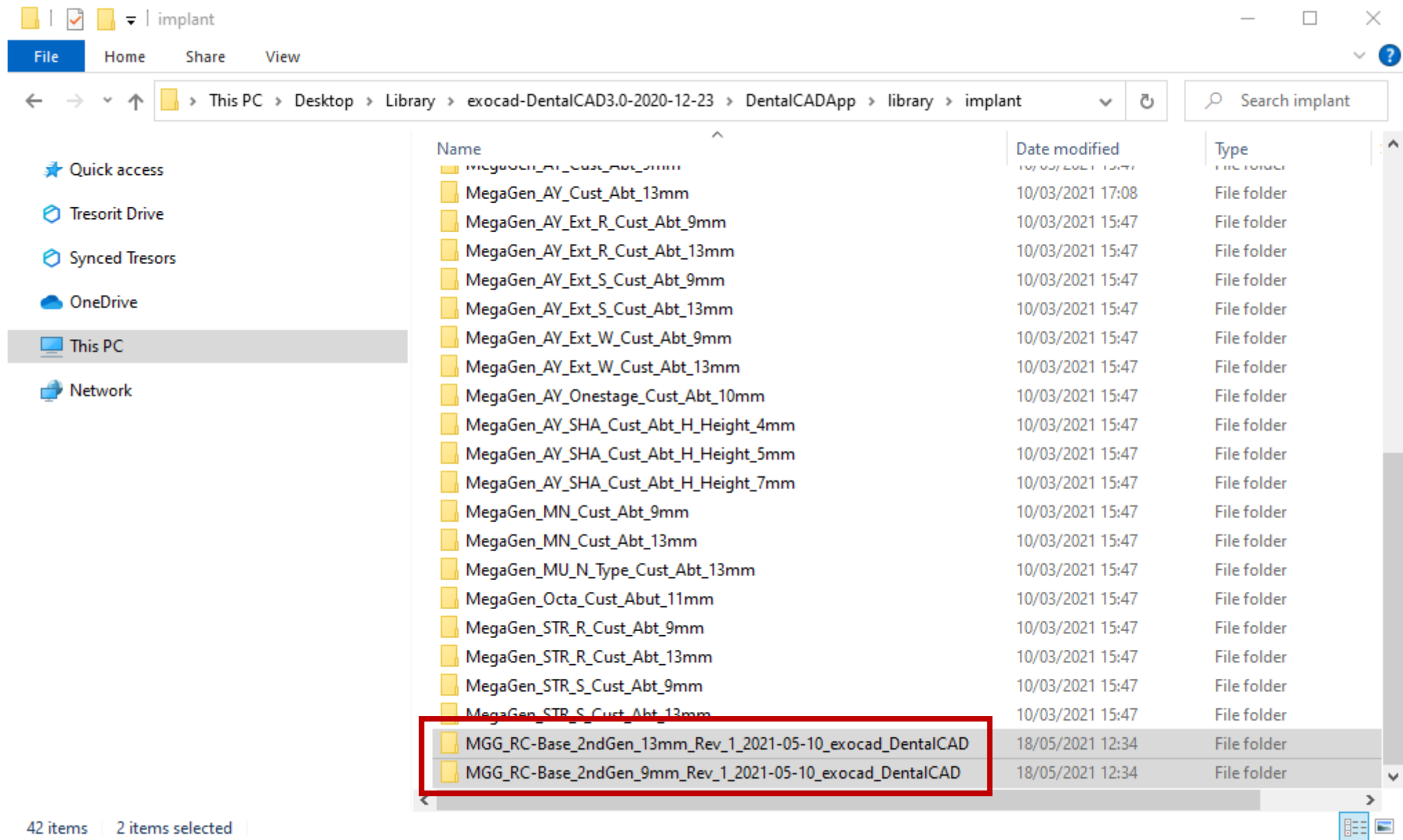

IMPORTANT: do not rename the folders otherwise the libraries will not work!


Go back to the folder **library** and open the folder **modelcreator**

The screenshot shows a Windows File Explorer window with the address bar path: This PC > Desktop > Library > exocad-DentalCAD3.0-2020-12-23 > DentalCADApp > library. The left sidebar shows 'This PC' selected. The main pane displays a list of folders with columns for Name, Date modified, Type, and Size. The 'modelcreator' folder is highlighted and circled in red.

Name	Date modified	Type	Size
articulator	27/12/2020 21:08	File folder	
artiregister	27/12/2020 21:09	File folder	
attachments	01/10/2020 15:29	File folder	
bar	15/12/2020 09:24	File folder	
bolts	23/01/2020 09:50	File folder	
bridgesplitter	23/01/2020 09:50	File folder	
controls	27/12/2020 20:35	File folder	
gfx	27/12/2020 21:09	File folder	
implant	09/07/2021 09:53	File folder	
metadata	27/12/2020 21:09	File folder	
modelcreator	27/12/2020 21:11	File folder	
movementregister	23/01/2020 09:54	File folder	
nesting	23/01/2020 09:54	File folder	
ortho	27/12/2020 21:06	File folder	
pontics	27/12/2020 21:09	File folder	
production	27/12/2020 21:09	File folder	
prosthetictoothpresets	21/12/2020 16:24	File folder	
prosthetictoothsets	03/05/2021 10:44	File folder	
rendereffects	27/12/2020 20:35	File folder	
retentions	23/01/2020 09:55	File folder	
smiledesign	27/12/2020 20:35	File folder	
surgicalguide	03/05/2021 10:44	File folder	

24 items | 1 item selected

Open the folder implants

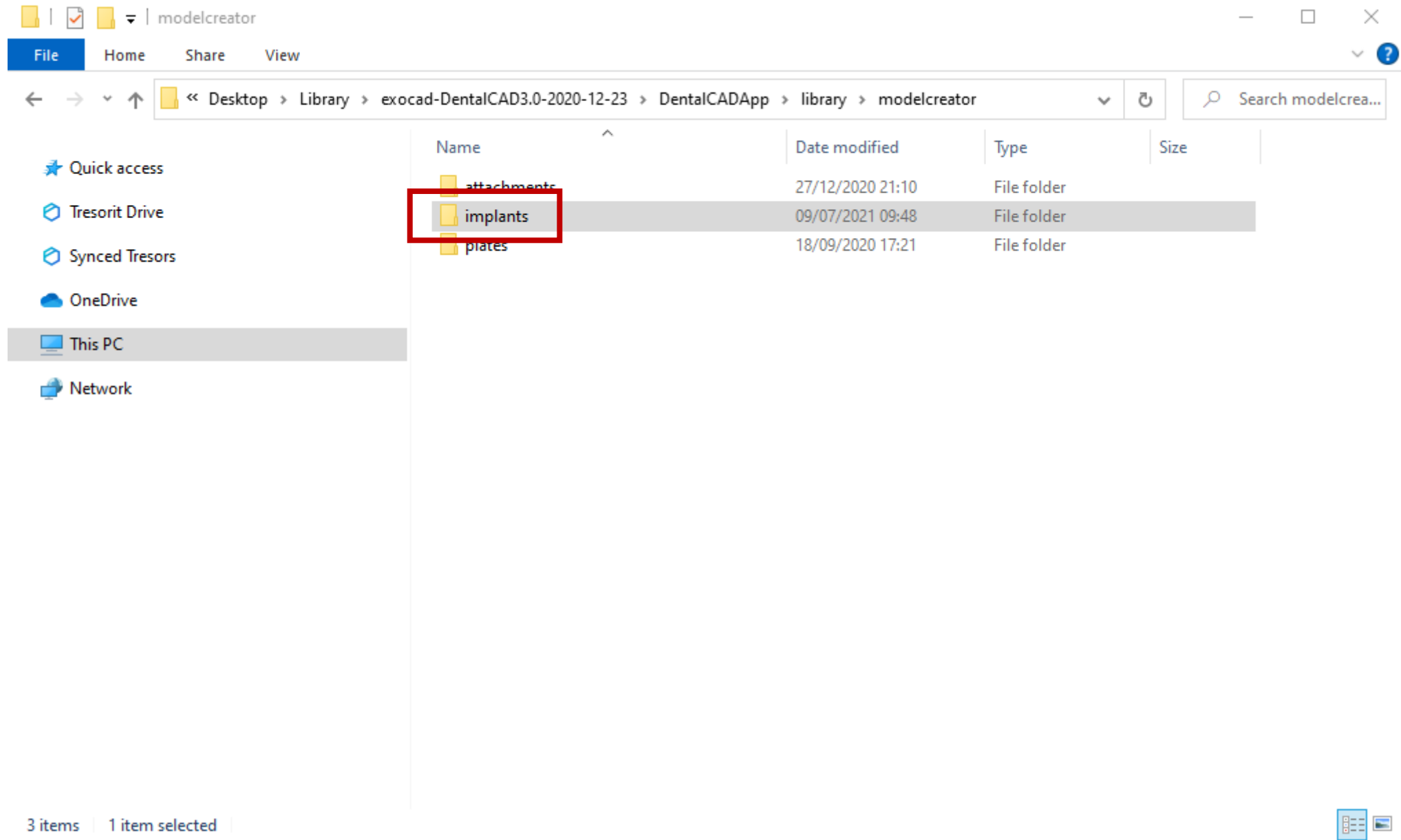

Leave the folder **implants** opened


File Explorer window showing the path: Library > exocad-DentalCAD3.0-2020-12-23 > DentalCADApp > library > modelcreator > implants

Left sidebar: Quick access, Tresorit Drive, Synced Tresors, OneDrive, **This PC**, Network

Name	Date modified	Type	Size
exocad_Demo_Case_ModelCreator_Library	27/12/2020 21:11	File folder	
exocad_Demo_ModelCreator_Library	27/12/2020 21:11	File folder	
MegaGen_AR_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_Extra_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_Octa1_N_R_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_Octa1_SHA_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_SHA_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_Ext_S_R_W_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_Onestage_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_SHA_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_MN_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_MU_N_type_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_Octa_S_R_W_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_STR_S_R_Cus_Abt_RP	03/05/2021 11:11	File folder	

15 items

Move the folder containing the ModelCreator library...

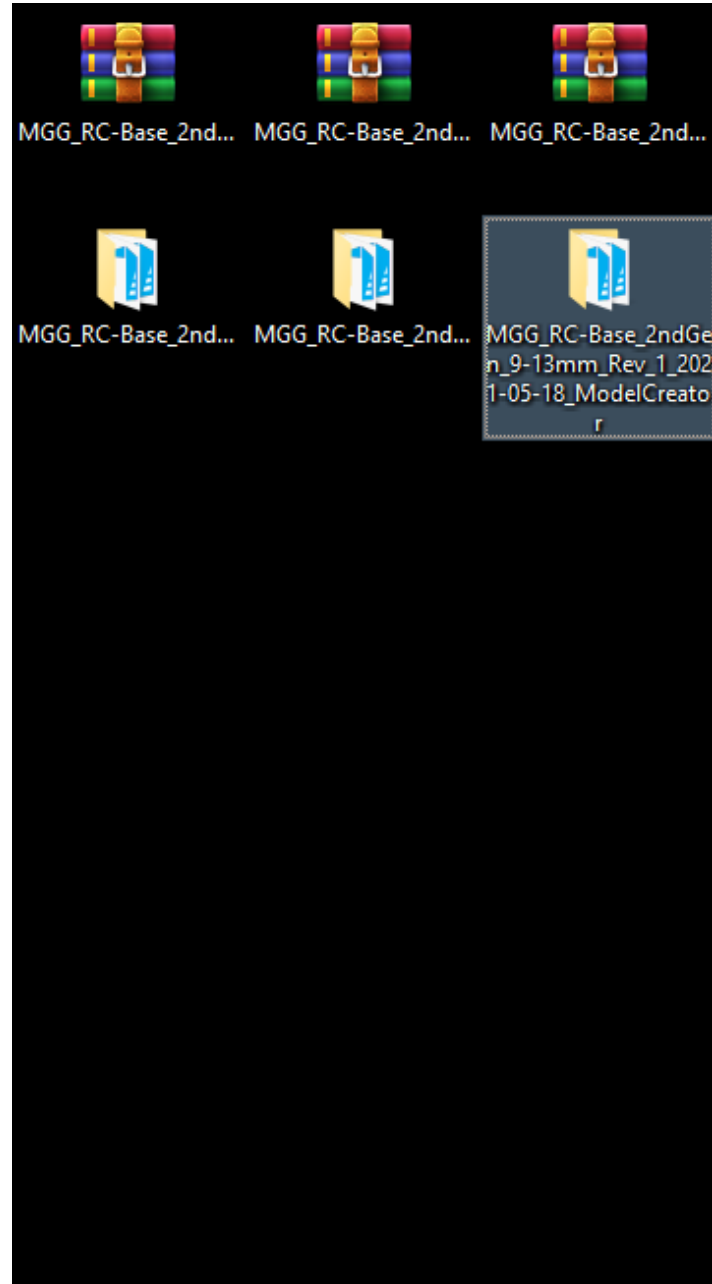

... in the **implants** folder left opened before

The screenshot shows a Windows File Explorer window titled 'implants'. The address bar shows the path: Library > exocad-DentalCAD3.0-2020-12-23 > DentalCADApp > library > modelcreator > implants. The left sidebar shows 'This PC' selected. The main pane displays a list of 16 folders with columns for Name, Date modified, Type, and Size. The folder 'MGG_RC-Base_2ndGen_9-13mm_Rev_1_2021-05-18_ModelCreator' is highlighted with a red box.

Name	Date modified	Type	Size
exocad_Demo_Case_ModelCreator_Library	27/12/2020 21:11	File folder	
exocad_Demo_ModelCreator_Library	27/12/2020 21:11	File folder	
MegaGen_AR_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_Extra_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_Octa1_N_R_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_Octa1_SHA_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AR_SHA_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_Ext_S_R_W_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_Onestage_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_AY_SHA_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_MN_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_MU_N_type_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_Octa_S_R_W_Cust_Abt_RP	03/05/2021 11:11	File folder	
MegaGen_STR_S_R_Cus_Abt_RP	03/05/2021 11:11	File folder	
MGG_RC-Base_2ndGen_9-13mm_Rev_1_2021-05-18_ModelCreator	18/05/2021 16:29	File folder	

IMPORTANT: do not rename the folders otherwise the libraries will not work!